

When printing, please make sure your printer settings are set to print "Actual Size" to ensure the ruler on the sizing guide is accurate.


Aspen® Cervical Collar Sizing Guide

PICK THE RIGHT SIZE



With the head in the neutral or desired position, place the guide on the highest point of the trapezius and flat against the side of the head.

Sight an imaginary perpendicular line from the bottom of the chin back to the guide and select the proper size Aspen Collar.

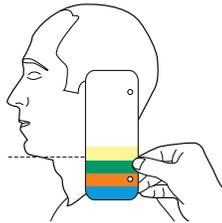

CHECK FOR A GOOD FIT

1. The chin must be flush with the front of the chin piece plastic.
2. The FlexTabs™ on the front and back panels should be flexed.
3. The collar should be "snug" but not touch the tracheal area.

EXTRA TALL

TALL

REGULAR

SHORT

Proper training in the use of this device should take place before it is applied. These directions are guidelines only and are not offered as medical recommendations.

Aspen® Pediatric Line Sizing Guide

A PROPER FIT AS EASY AS 1-2-3

1. **Chin-Dots hidden?** The chin must be flush with the front of the chin piece plastic.
2. **Velcro® past mark?** Back panel plastic must overlap the sides of the front panel.
3. **FlexTabs™ bent?** The FlexTabs™ on the front and back panels should be flexed.



SIZING GUIDELINES

PD 1		1-18 months
		weight 11-29 lbs (5-13 kg) length 21-33 in (53-84 cm)
PD 2		9-24 months
		weight 22-33 lbs (10-15 kg) length 29-37 in (74-94 cm)
PD 3		1-3 years
		weight 24-36 lbs (11-16 kg) height 33-40 in (84-102 cm)
PD 4		2-5 years
		weight 26-42 lbs (12-19 kg) height 35-45 in (89-114 cm)
PD 5		3-6 years
		weight 27-54 lbs (12-25 kg) height 37-48 in (94-122 cm)
ADULT Short		5 years and up
		weight 36 lbs (16 kg) and up height 42 in (107 cm) and up



Please Refer to Pediatric Collar Instruction Sheet for More Information.

Aspen MEDICAL PRODUCTS © Aspen Medical Products. aspenmp.com
800-295-2776 • 949-681-0200 LIT5018D